# ACTIVA™ BioACTIVE Bulk Flow™

ACTIVA™ BioACTIVE Bulk Flow™  ShadeFusion™

ACTIVA™ KIDS BioACTIVE Bulk Flow™  ShadeFusion™  
& Opaque White (Pediatric) Shade

Bulk Fill – Self-leveling – Fluoride, Calcium & Phosphate –  
Dual Cure – Moisture Tolerant – BPA Free

## PRODUCT DESCRIPTION

ACTIVA™ BioACTIVE dual-cure materials are the first dental restoratives with a bioactive polymer matrix (ION-X™), patented anti-stress & shock-absorbing functioning elastomer (MODULUS™), and hybridized reactive ionomer glass designed to mimic the physical and chemical properties of natural teeth and enable long-term release & recharge of fluoride, calcium, and phosphate. They feature moisture-tolerant WETBOND™ technology and contain no Bisphenol A, no Bis-GMA, and no BPA derivatives.

Self-leveling characteristics of ACTIVA™ BioACTIVE Bulk Flow™ make it ideal for maximum marginal adaptation without additional manipulation, while it remains thixotropic enough to remain in preparations. ACTIVA BioACTIVE Bulk Flow has an unlimited depth of cure, allowing for a one-step “bottom-to-top” bulk-fill placement following the initial preparation/etch/bond. It can also be used as a dentin replacement material, paired with a capping layer of universal composite. ACTIVA BioACTIVE Bulk Flow and ACTIVA™ KIDS BioACTIVE Bulk Flow™ feature ShadeFusion™ chroma and translucency matching technology. ACTIVA Bulk Flow Universal Shade replaces most commonly used VITA shades, with the best aesthetic chameleon coverage of adult shades in the range of A2-A4. ACTIVA KIDS Bulk Flow Universal Shade is designed to produce a highly aesthetic chameleon coverage of pediatric shades (B1-A1). ACTIVA KIDS Bulk Flow is also available in a more opaque white option (Opaque White Shade).

## INDICATIONS FOR ACTIVA BioACTIVE BULK FLOW

ACTIVA Bulk Flow is designed as a self-leveling flowable for dentin and enamel replacement. It is intended to be used in restoring large class Is and IIs and is also suitable as a bioactive filling material for pits, root surface cavities, and all other classes of restorations.

## CONTRAINDICATIONS FOR ACTIVA BioACTIVE BULK FLOW

Not indicated for direct placement on the exposed pulp. See “Instructions for Use” section below. Do not use on patients with known allergies or hypersensitivities to the product. Refer to the MSDS for complete chemical composition.

## HOW TO USE THE AUTOMIX SYRINGE

- Remove cap. If necessary, bleed the syringe so the base and catalyst are at the orifice of the syringe barrels. Place a mixing tip on the automix syringe. (Note: ACTIVA BioACTIVE Bulk Flow comes with 19-gauge, high-efficiency brown automix tips. 20-gauge brown automix tips are available from Pulpdent if a more precise placement technique is desired.)
- To ensure an even mix of base and catalyst, dispense 1–2mm onto a pad and discard this material.
- Dispense material directly onto the tooth or into the restoration.
- Discard mixing tip. Recap syringe. Do not cross-contaminate base and catalyst during recapping.

## APPLY ACTIVA TO A DRY TOOTH SURFACE, BUT DO NOT DESICCATE THE TOOTH

Using high-volume evacuation, compressed air, and/or a cotton pellet, dry and remove all surface moisture from the prepared tooth. Do not desiccate the tooth, which naturally contains a small amount of water. Ceramic, metal, resin-desensitized, prehybridized, and cured composite surfaces should be dry.

## ACTIVA BioACTIVE BULK FLOW INSTRUCTIONS FOR USE

- Set curing light to a 20-second, low-intensity setting (10 seconds if using a mid-/high-powered [1200 mW/cm<sup>2</sup>] curing light).
- Isolate and prepare tooth to receive a restoration. Ideal margin preparations are rounded with no sharp angles. In Class V lesions, bevel or undercut enamel.
- Place appropriate pulp protection, if indicated.
- Apply a bonding agent of your choice in accordance with the manufacturer’s instructions for use. ACTIVA BioACTIVE Bulk Flow is compatible with total-etch, self-etch, and selective-etch bonding systems and techniques. If total etching, etch the prepared tooth with 37–38% phosphoric acid. Etching time is 20 seconds for enamel and 15 seconds for dentin. Selective etch enamel if desired. After etching, rinse and lightly dry, removing all excess moisture with high-volume evacuation, compressed air, and/or a cotton pellet, but do not desiccate the tooth.
- Place an automix tip on the ACTIVA syringe. Dispense material using gentle pressure. To ensure an even mix of base and catalyst, dispense 1–2mm of material onto a mixing pad and discard. ACTIVA is air-inhibited and will not set on the pad in the presence of oxygen.
- ACTIVA is a dentin and enamel replacement material. Place mixing tip at cavity floor. Apply ACTIVA in bulk fill (unlimited depth) or increments (layering), keeping mixing tip submerged in the material. Light cure for 20 seconds (10 seconds with medium-/high-powered curing light) after placement is complete (bulk fill) or between each layer. Add a capping layer of universal composite if used as dentin replacement. Anaerobic self-cure setting time at mouth temperature is 3–5 minutes. If allowing to self-cure, cover exposed ACTIVA surfaces with an oxygen-inhibiting barrier, e.g., glycerin. Finish and polish in the usual manner.
- ACTIVA can also be used with both open and closed sandwich techniques.

**Optional:** Allow up to 10 seconds for ACTIVA to self-level and begin to self-cure. This mitigates polymerization stress and exothermic reaction while material fully adapts to the cavity. Do not exceed manufacturer’s recommended curing time. Additional curing may generate excessive heat, damaging the pulp.

**Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

## CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves. Do not exceed manufacturer’s recommended curing time for the light you are using.

## STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination, and sources of ignition.
- Re-cap immediately after use.

## REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262 / [pulpdent.com](mailto:pulpdent.com) / email: [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

**Note:** Refer to the Safety Data Sheet for a list of materials present in this product.

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## About ACTIVA™ BioACTIVE Dual-Cure Products

**Moisture Tolerant • Dual Cure • Ion-Releasing • Radiopaque**  
**Contains No Bisphenol A, No Bis-GMA, No BPA derivatives**

ACTIVA™ BioACTIVE dual-cure products are strong, durable, ionic restorative resins that have the esthetics and physical properties of composites.<sup>1,2</sup>

They release fluoride, calcium, & phosphate, and have a greater potential to recharge these minerals than glass ionomers and traditional RMGIs. ACTIVA stimulates mineral apatite crystal formation at the material-tooth interface.<sup>3,4</sup> This natural remineralization process knits together the restoration and the tooth, penetrates and fills micro-gaps, and seals margins against microleakage.

ACTIVA products react to pH changes in the oral environment. They actively participate in the ionic exchange with saliva and tooth structure that is essential for maintaining healthy teeth, and they continuously recharge the ionic components of saliva, teeth, and the material itself. For this reason, ACTIVA is called a “smart” material.

Unlike traditional materials that are hydrophobic, repel water, and are designed to be passive, ACTIVA is moisture tolerant and plays a dynamic role in the mouth. Only moisture-tolerant materials that are partly water-based or have the capacity for significant water transport or storage can react to changes in the ambient conditions and are capable of this dynamic behavior?<sup>5</sup>

ACTIVA dual-cure materials contain water, yet the material has very low solubility.<sup>6</sup> The ionic resin matrix facilitates the diffusion of fluoride, calcium, & phosphate ions while still maintaining the excellent physical properties associated with resins and composites.

The resin matrix displays exceptional marginal integrity, sealing ability against marginal leakage, and intimate adaptation to tooth structure.<sup>4,7,8</sup> It contains an acidic monomer that improves the interaction between the resin component and the glass ionomer and enhances the interaction with tooth structure.

ACTIVA BioACTIVE dual-cure products are two-paste systems in automix syringes. They have three setting mechanisms: light cure, self-cure resin chemistry, and self-cure glass ionomer reaction.

#### References

- Gim V, et al. J Dent Res 93 (Spec Iss A) 1163, 2014 ([www.iadr.org](http://www.iadr.org)).
- Garcia- Godoy F, Morrow BR. J Dent Res 94 (Spec Iss A) 3522, 2015 ([www.iadr.org](http://www.iadr.org)).
- Chao W, et al. J Dent Res 95 (Spec Iss A) S 1313, 2016 ([www.iadr.org](http://www.iadr.org)).
- Garcia-Godoy F, Morrow BR. J Dent Res 95 (Spec Iss A) 1828, 2016 ([www.iadr.org](http://www.iadr.org)).
- McCabe JF, et al. Aust Dent J 2011 Jun;56 Suppl 1:3-10.
- Hall J, et al. J Dent Res 95 (Spec Iss A) SI 126, 2016 ([www.iadr.org](http://www.iadr.org))
- Zmner, Osvaldo, Cornelis H, Pameijer, and Sandra Hernandez. "Resistance against bacterial leakage of four luting agents used for cementation of complete cast crowns." *Am J Dent* 27.1 (2014): 51-5 .
- Kane B, et al. Am J Dent 2009;22(2):89-91.

#### EU Importers

<p><b>Leibetseder Dentalwarenhandel GmbH</b> Hans-Zach-Straße 2 4210 Gallneukirchen Austria</p> <p><b>Bohumila Sieglová</b> Kolná 303 25203 Řitka Czech Republic</p> <p><b>Hu-Fa Dental a.s.</b> Moravni 909 765 02 Otrokovice Czech Republic</p>	<p><b>Cenger Scandinavia A/S</b> Urlevvej 6 B 8783 Hornslyd Denmark</p> <p><b>Dansk Nordenta A/S</b> Nydamsvvej 8 8362 Hørring Denmark</p> <p><b>Plandent OY</b> Asentajankatu 6 B 00880 Helsinki Finland</p> <p><b>OPSYSE</b> 18 Allée Du Fenouil ZA Saint Louis 84250 Le Thor France</p>	<p><b>Kinderdent GmbH</b> Gutenbergsstraße 7 28844 Weyhe Germany</p> <p><b>Ortho Organizers GmbH</b> Lauenbühlstraße 59 88161 Lindenberg Im Allgäu Germany</p> <p><b>Dental Medical Ireland</b> Kellystown House Kellystown Lane, Leixlip Co. Kildare, W23 V9p Ireland</p>	<p><b>Ravelli S.p.A.</b> Via Darwin 32-34 20019 Settimo Milanese Italy</p> <p><b>Virdental, UAB</b> Artojų G. 52b, Garliavos Apylinkių Seniūnija Teleičių Kaimas 53273, Kauno Rajonas Lithuania</p> <p><b>Hofmeester Dental B.V.</b> Andries Copierhof 2 3059 Lm Rotterdam Netherlands</p>	<p><b>Equadent Sp z.o.o.</b> Ul. Pck 12 62-500 Konin Poland</p> <p><b>Lifco Dental AB</b> Verkmästaregatan 1 745 39 Enköping Sweden</p> <p><b>UNIDENT AB</b> Västerhavsvägen 2 311 77 Falkenberg Sweden</p>
---	--	--	--	---

Electronic Safety Data Sheet	
EN	Electronic Safety Data Sheet
ES	Ficha electrónica de datos de seguridad
FR	Fiche de données de sécurité électronique
DE	Elektronisches Sicherheitsdatenblatt
IT	Scheda di sicurezza elettronica
DK	Elektronisk sikkerhedsdatablad
NL	Elektronisch veiligheidsinformatieblad
SE	Elektroniskt säkerhetsdatablad
NO	Datablad om elektronisk sikkerhet
FI	Sähköinen käyttöturvallisuustiedote
CZ	Elektronicky bezpečnostní list
LV	Elektroniskā drošības datu lapa
LT	Elektroninis saugos duomenų lapas
PT	Ficha de dados de segurança eletrónica
SK	Elektronická karta bezpečnostných údajov
PL	Elektroniczna karta charakterystyki
RO	Fișa electronică cu date de securitate
EE	Elektroniline ohutuskaart
BG	Електронни информационен лист за безопасност
GR	Ηλεκτρονικό δελτίο δεδομένων ασφαλείας

**SCAN ME**

[www.pulpdent.com/info](http://www.pulpdent.com/info)

## PULPDENT® Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.  
 Tel. (617) 926-6666 / (800) 343-4342 / Fax (617) 926-6262  
[pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com) • [pulpdent.com](http://pulpdent.com)