PULPDENT

PerioCare® Periodontal Dressing

PRODUCT DESCRIPTION

PerioCare is a highly elastic periodontal dressing that sets resiliently hard, does not chip, and provides durable protection of tissue.

INDICATIONS

Post-surgical periodontal dressing for protection of soft tissues.

CONTRAINDICATIONS

Not for use on cold sores.

DIRECTIONS FOR USE

Helpful Hint: To accelerate setting time, add 1-2 drops of water to PerioCare® paste and gel and incorporate water into the mix.

1. Extrude equal amounts of paste (tube 1) and gel (tube 2) onto mixing pad.
2. Use a spatula to thoroughly mix paste and gel until color becomes uniform throughout.
3. Form PerioCare mixture into log shape on mixing pad.
4. Test for tackiness with spatula or wet finger. PerioCare is ready to use with moistened fingers in about 60-90 seconds.
5. Mold PerioCare into desired shape using fingers. PerioCare will remain workable for approximately 4-5 minutes. If additional working time is required, place PerioCare in hot water to soften.
6. Clean spatula and instruments immediately with Wonder Orange™ solvent or with alcohol.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Pulpdent periocare Periodontal dressing

Rx only

- Easy to measure: use equal amounts of paste and gel.
- Easy to mix: paste and gel combine easily and completely.
- Complete: no additional ingredients are necessary.
- Ready to use quickly: 60–90 seconds
- Readily formed into cords of any thickness or length for easy application in the mouth.
- Cohesive: PerioCare melds to itself during working time.
- Adhesive: PerioCare stays in the mouth as long as needed without shrinkage or disintegration.

- Easily removed.
- Naturally colored.
- Neutral aroma and taste.
- Contains no eugenol.
- The package is designed for efficient and convenient office storage. Each tube is clearly marked to avoid confusion between paste and gel.
- Economical for the dentist. Package contains 6 fl. oz. (177.4 ml). A mixing pad is included in the PerioCare kit.

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Hofmeister Dental B.V.
Lifco Dental AB
UNIDENT AB
PULPDENT® Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.