embrace™ wetbond™ pit & fissure sealant


fluoride releasing - light cure - resin based ■ Contains No Bisphenol A

Embrace bonds chemically and micromechanically to slightly moist teeth. It is tooth integrating and margin-free, providing an exceptional seal against microleakage.


ALWAYS APPLY EMBRACE TO A SLIGHTLY MOIST TOOTH SURFACE.

XP-EMS-IN-18w
REV: 11/2025

QUICK REFERENCE GUIDE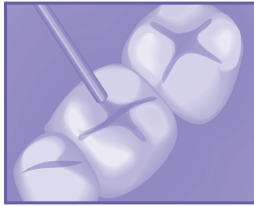

1. Clean, isolate and etch tooth for 20 seconds. Rinse well. Lightly dry. LEAVE TOOTH SURFACE SLIGHTLY MOIST.

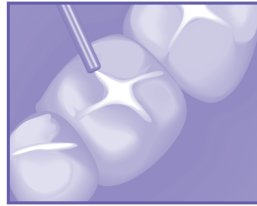2. Apply Embrace sealant on slightly moist occlusal surface.

3. Light cure. Embrace cures with all lights.

PRODUCT DESCRIPTION

Embrace WetBond Pit & Fissure Sealant is moisture friendly, resin-based sealant that releases fluoride and contains no Bis-GMA, no Bisphenol A, and no BPA derivatives.

INDICATIONS

For sealing pits and fissures of teeth

CONTRAINDICATIONS

Not for direct placement on the exposed pulp.
For any potential allergies, refer to associated MSDS.

INSTRUCTIONS FOR USE

Apply Embrace sealant to slightly moist tooth surfaces. Slightly moist surfaces exhibit neither dryness nor pooling of water. Lightly dry and remove surface water with compressed air or a cotton pellet, but do not desiccate the tooth, which naturally contains water. Surfaces should be shiny or glossy. Overly wet surfaces will result in decreased bond strength.

1. Thoroughly clean enamel surfaces with oil-free cleaning paste.
2. Rinse well, removing all residual paste from pits and fissures.
3. Prepare questionable enamel in the usual manner.
4. Isolate treatment area and dry lightly with oil-free compressed air.
5. Apply Etch-Rite 38% phosphoric acid etching gel to the clean tooth surface for 20 seconds. Rinse well. Do not disturb this surface.
6. Lightly dry and remove surface water with a cotton pellet or clean compressed air, but do not desiccate the tooth. Leave tooth surfaces slightly moist. Slightly moist surfaces should appear shiny or glossy.
7. Embrace bonds to surfaces slightly moist from saliva; however, it is best to avoid bacterial contamination.
8. Place a 22-gauge or 23-gauge applicator tip securely on Embrace syringe and carefully start flow of sealant. Use a new applicator tip for each patient. Recap syringe after use.
9. Apply Embrace to the prepared tooth surface. Flow the material from cusp to cusp, but do not cover the marginal ridges.
10. Light cure.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multi-use-dental-dispenser-devices>

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves. Do not exceed manufacturer's recommended curing time for the light you are using.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Re-cap immediately after use.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body.

Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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