**PULPDENT®**

# Forendo™ Paste

**Calcium hydroxide with Iodoform**

**Radiopaque – pH>12**

**XP-FOR-IN-07W**

**REV: 02/2023**

**© 2023 Pulpdent Corporation, All Rights Reserved.**

### PRODUCT DESCRIPTION

Forendo Paste is a temporary or permanent endodontic filling material indicated for routine use as a root canal dressing between office visits; for treatment of abscesses, periapical lesions, perforations, exudation, root resorption, traumatic injuries; and for apexification and apexogenesis. Forendo Paste can be used alone as a temporary canal treatment for vital and non-vital pulpotomies, and for pulpectomies in deciduous teeth.

Forendo Paste is a non-setting, premixed paste that contains calcium hydroxide and iodoform in a silicone oil base. It has a pH>12 and is radiopaque.

Forendo Paste is not intended for use as a root canal filling material in root canal restorations, root canal obturation, or root canal irrigation.

### INDICATIONS

Root canal therapy

### CONTRAINDICATIONS

Not a hard-setting root canal filling material

### INSTRUCTIONS FOR USE

#### TEMPORARY CANAL TREATMENT

1. Using good endodontic procedures, cleanse and prepare the root canal in the conventional manner. Dry the canal.

2. Place a new disposable applicator tip securely on the luer lock syringe.

3. Fill the canal by applying gentle pressure on the syringe plunger. Do not force the plunger. The paste can also be placed into the canal with a lentulo spiral.

4. Remove excess at the orifice with a cotton pellet.

5. Seal access opening with your preferred cement.

6. Discard applicator tip and cap syringe immediately after use.

7. Forendo Paste is easily removed with file and irrigation with sodium hypochlorite.

#### TREATMENT OF PRIMARY TEETH

1. Follow instructions above for Temporary Canal Treatment.

2. It is not necessary to remove Forendo Paste from the root canal. The material will resorb with the roots in deciduous teeth.

#### STORAGE AND HANDLING

• When dispensing from the syringe, do not force plunger. Apply light pressure only. If material does not dispense, remove needle, purge some material from the syringe, and place a new needle on the syringe.

• Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.

• Shelf life of unopened product: 2 years from date of manufacture.

• Re-cap syringe immediately after use.

### CAUTION

For professional use only by dentists. Wear protective eyewear and surgical gloves. Avoid contact with eyes, mucous membranes and skin. If contact occurs, flush with copious amount of water. If contact occurs with eyes, flush with copious amounts of water and seek medical attention immediately.

#### REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

**Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: **<https://www.fda.gov/medical-devices/dental-devices/multi-ple-use-dental-dispenser-devices>**

**Note:** Refer to the Safety Data Sheet for a list of materials present in this product.

**© 2023 Pulpdent Corporation, All Rights Reserved.**

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**Hofmeester Dental B.V.**
**Lifco Dental AB**
**UNIDENT AB**
#### *PULPDENT Corporation*
80 Oakland Street • Watertown, MA 02472 •U.S.A.