Dry-Rite™ Drying Agent

Contains Ethyl Alcohol

For use in appropriate clinical situations where residual moisture may effect bonding or cementation.

PRODUCT DESCRIPTION

Dry-Rite Drying Agent is a preparation of anhydrous ethyl alcohol with an organic solvent.

INDICATIONS

For use in appropriate clinical situations where residual moisture may affect bonding or cementation.

CONTRAINDICATIONS

Not for placement on dentin or soft tissue.

DIRECTIONS FOR USE

1. Remove syringe cap and place a 22-gauge red dropper tip on the syringe.
2. Isolate the operative area in the mouth to maintain a clean dry field.
3. Rinse the operative area and air dry with clean, uncontaminated, oil-free and moisture-free compressed air.
4. Dispense a small amount of Dry-Rite on the desired surface. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between forefinger and thumb while resting the plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dropper tip for each patient.
5. To promote rapid drying, use a gentle flow of clean, uncontaminated air, if desired.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool

room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.

- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

CAUTION

Dry-Rite with ethyl alcohol is flammable. Handle with care and keep away from open flame. Do not place Dry-Rite directly on dentin as this can desiccate the dental tubules.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262
email: pulpdent@pulpdent.com

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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